


CrossFix® II


All-Suture, All-Inside Meniscal Repair System

Surgical Technique

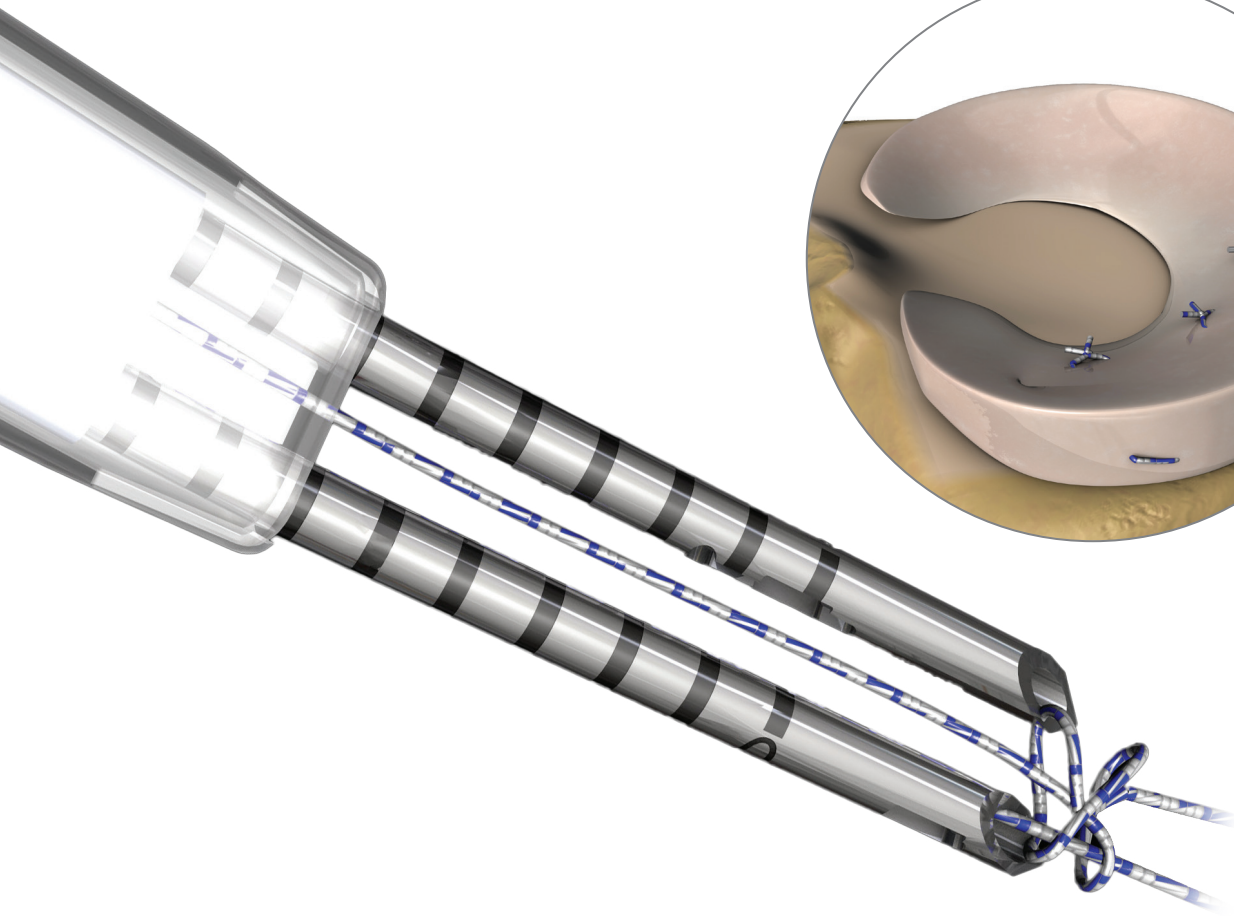


All-Inside Repair with **Inside-Out Results**

The CrossFix II Meniscal Repair System is a suture only device that creates an instantaneous mattress stitch for a **fast, strong, and reliable** all-inside meniscal repair.

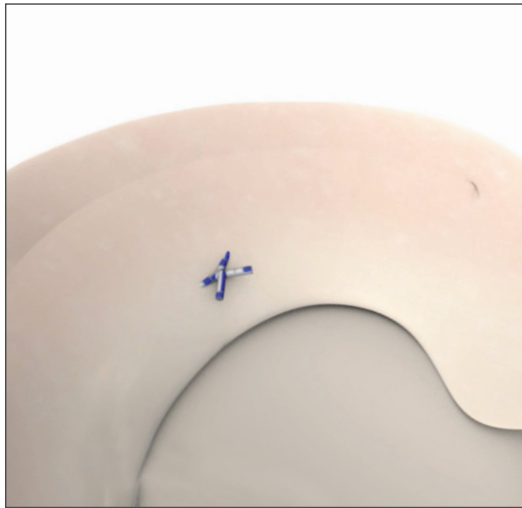


Fast

Single insertion of device into meniscus with pre-tied sliding knot creates an instantaneous 3 mm mattress stitch.

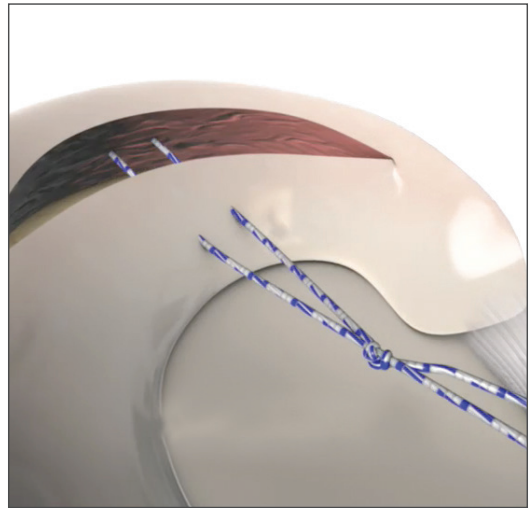

Strong

Biomechanically strong fixation with pull - out forces analogous to open suture techniques.*

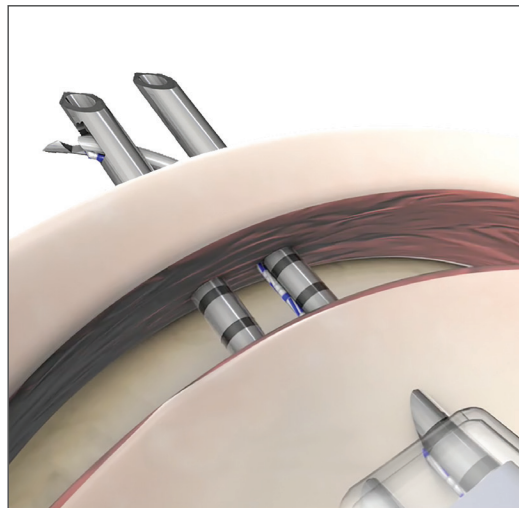

Reliable

Consistent deployment of suture in both straight and curved dilating needle options. Reproducible technique without fiddle factor.

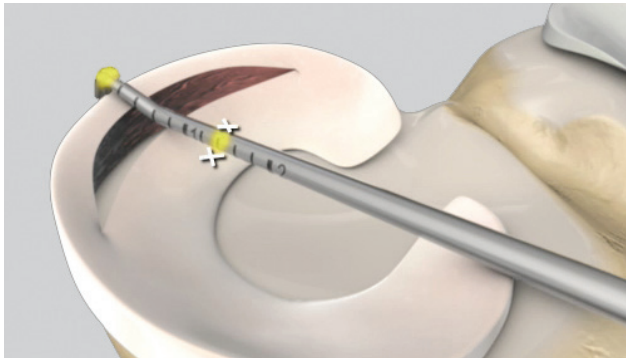


Figure 1



Figure 3



Figure 2

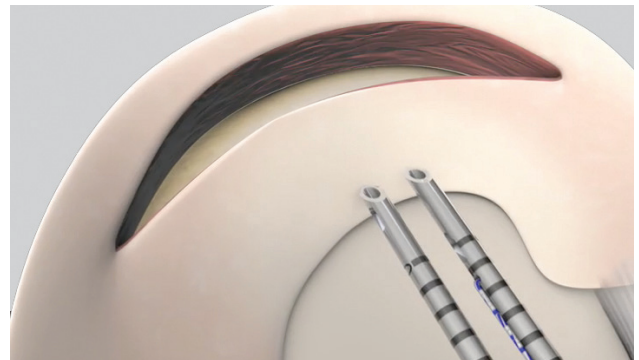


Figure 4

Step 1

Access the location of the meniscal tear and determine the reparability. If the tear is repairable, determine appropriate portal placement for instrumentation. Using the Probe in the CrossFix II Disposable Kit, measure the tear (Figure 1).

ⓘ **Note:** The tip of the Probe can be bent to the desired angle by using hemostats or other instrumentation.

Step 2

Set the depth limiter to the desired distance by pulling the black lever on the handle of the CrossFix II device backwards (Figure 2).

ⓘ **Note:** This is done outside of the knee joint.

Step 3

Insert the Introducer (from the Disposable Kit) through the portal first, and then slide the exposed CrossFix II needles along the Introducer into the joint space (Figure 3).

ⓘ **Note:** This helps the CrossFix II pass through the tissue and into the joint space without torquing the needles.

Step 4

Once the needles are in the joint space, remove the Introducer from the knee (Figure 4).

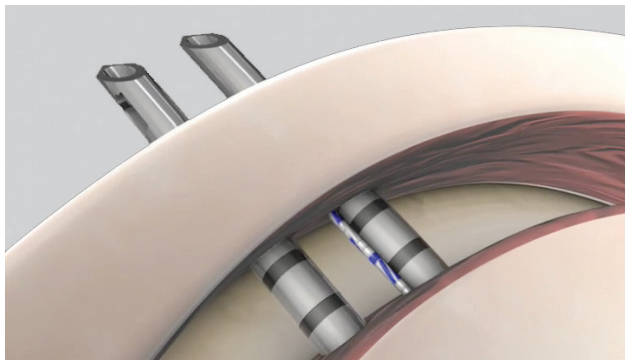


Figure 5

Step 5

Insert the needles into meniscus capturing the tear. The needles will be in position when the depth limiter pushes against the meniscal surface, limiting advancement (Figure 5).

- ⊖ **OR Tip:** To avoid needle torque, let go of the CrossFix II device after insertion through the meniscal tissue to allow the needles to re-align.
- ⊖ **OR Tip:** Lightly mallet CrossFix II into the knee space to ensure needles enter evenly.

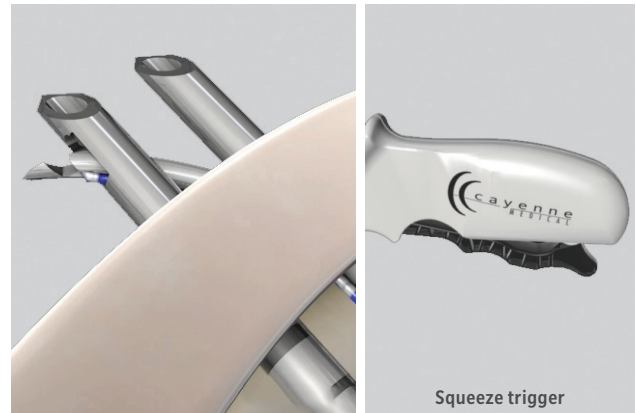


Figure 6

Figure 7

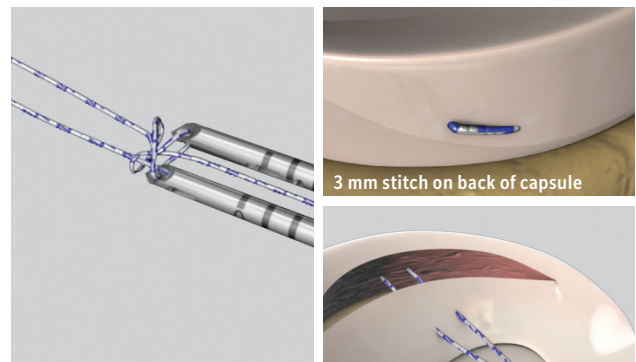


Figure 8

Step 6

With the needles in position, squeeze the device trigger, deploying the needle and passing the suture. The trigger can be released after hearing an audible click (Figure 6 and 7).

- ⊖ **Note:** A 20° oblique stitch is biomechanically equivalent to a vertical stitch.

Step 7

Remove the needles from the knee and gently pull the device out of the joint allowing the pre-tied sliding knot to advance into position (Figure 8).

- ⊖ **Note:** You will feel tension as the knot slides off the needles.

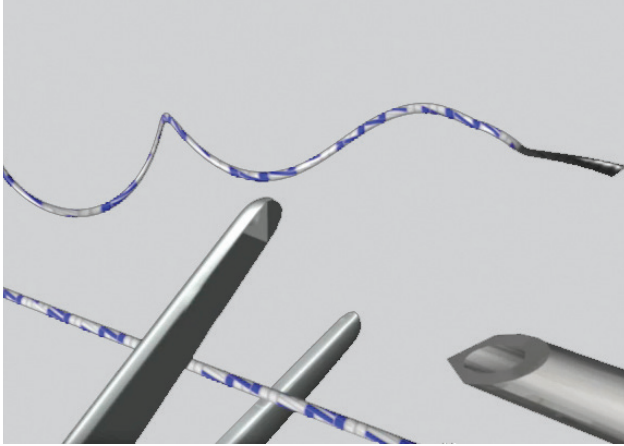


Figure 9

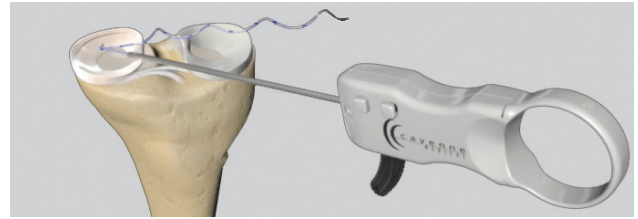


Figure 11

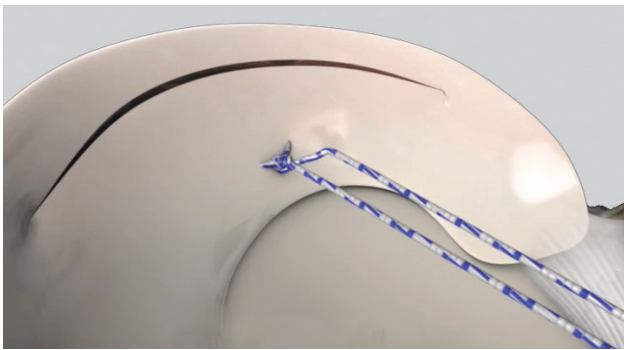
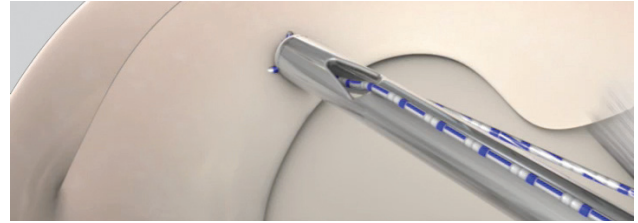


Figure 10

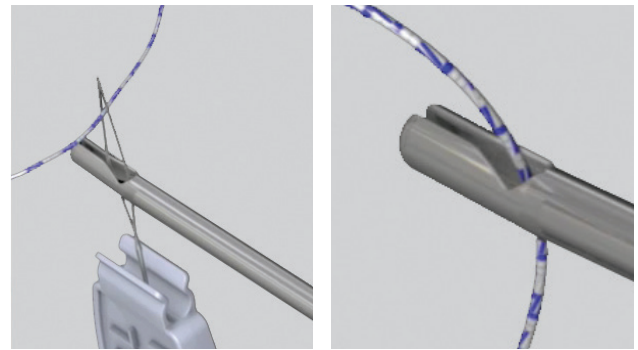


Figure 12

Step 8

The locking curly (black) end will fall off the needle as the device is slowly removed from the knee space. Cut the post (white) end off the needle tip (Figure 9).

ⓘ **Note:** Be careful not to cut or prematurely pull the black end of the suture as this results in permanently locking the knot.

Step 9

Pull the post (white) end of the suture to advance the sliding knot to reduce the meniscal tear (Figure 10).

Optional

If not satisfied with the reduction of the meniscal tear, thread the post (white) end of the suture through the CrossFix II Knot Pusher/Suture Cutter and apply tension to the post end (Figure 11).

ⓘ **Note:** Suture can be threaded through the Knot Pusher/Suture Cutter with packaged snare or by hand (Figure 12).

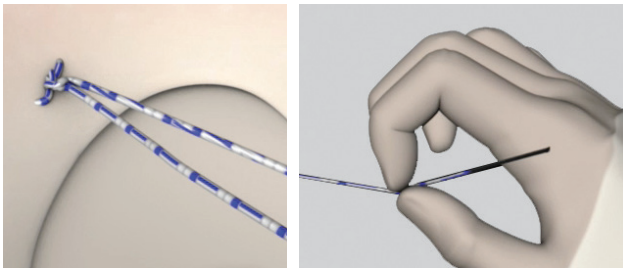


Figure 13



Figure 15

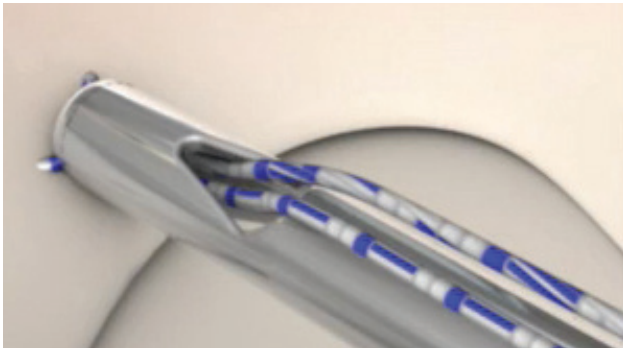


Figure 14

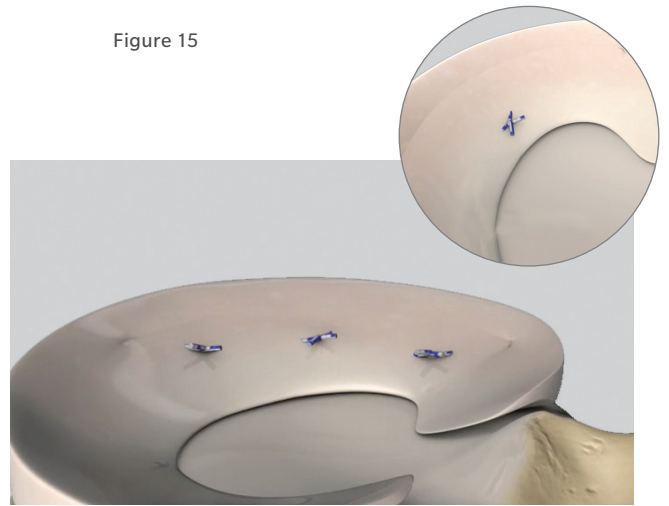


Figure 16

Step 10

With the knot in position, lock the knot by pulling on the curly black end of the suture. Half hitches can be added as needed (Figure 13).

Step 11

Thread both ends of the suture through the CrossFix II Knot Pusher/Suture Cutter. Rest the tip of the Knot Pusher/Suture Cutter on the knot to allow for a 2-3 mm tail (Figure 14).

Step 12

Cut the suture by pulling the black trigger backwards (Figure 15).

Step 13

Assess the fixation and repeat as needed (Figure 16).

Ordering Information

CrossFix II Meniscal Repair System (Sterile)

Description	Part Number
CrossFix II Disposable Procedure Kit (Probe, Knot Pusher/Suture Cutter, Introducer)	CM-8000
CrossFix II Meniscal Repair Device, Straight	CM-8001
CrossFix II Meniscal Repair Device, Up Curve	CM-8002

INDICATIONS FOR USE

The CrossFix II Meniscal Repair Device is intended for approximation of soft tissue in meniscal repair procedures.

CONTRAINDICATIONS

- Surgical procedures other than those listed in the INDICATIONS section.
- Presence of infection.
- Patient conditions including insufficient quantity or quality of tissue.
- Insufficient blood supply or previous infections which may hinder the healing process.
- Foreign body sensitivity. If material sensitivity is suspected, testing should be completed prior to suture implantation.
- Conditions which may limit the patient's ability or willingness to follow postoperative care instructions.

This material is intended for health care professionals. Distribution to any other recipient is prohibited. All content herein is protected by copyright, trademarks and other intellectual property rights owned by or licensed to Zimmer Biomet or its affiliates unless otherwise indicated. This material must not be redistributed, duplicated or disclosed, in whole or in part, without the express written consent of Zimmer Biomet.

Check for country product clearances and reference product specific instructions for use. For complete product information, including indications, contraindications, warnings, precautions, and potential adverse effects, see the package insert and www.zimmerbiomet.com.

Zimmer Biomet does not practice medicine. This document is intended for surgeons and is not intended for laypersons. Each surgeon should exercise his or her own independent judgment in the diagnosis and treatment of an individual patient, and this information does not purport to replace the comprehensive training surgeons have received. As with all surgical procedures, the technique used in each case will depend on the surgeon's medical judgment as the best treatment for each patient. Results will vary based on health, weight, activity and other variables. Not all patients are candidates for this product and/or procedure. Caution: Federal (USA) law restricts this device to sale by or on the order of a surgeon. Rx only.

©2018 Zimmer Biomet



Authorized Representative

Zimmer GmbH
Sulzerallee 8
8404 Winterthur
Switzerland



Legal Manufacturer

Cayenne Medical Inc.
16597 N. 92nd St
Suite 101
Scotsdale, AZ 85260
USA



ZIMMER BIOMET

Your progress. Our promise.®

1500.1-GLBL-en-REV0718

www.zimmerbiomet.com

0086